



Posey® Heel Pillow/Foot Positioner

Heel Pillow

The Posey Heel Pillow helps protect heels and ankles from skin irritation and painful pressure sores. The protectors are filled with soft polyester fiber, covered with a cotton fabric, and have a padded strap with hook and loop closures to provide a snug, yet comfortable fit. The Heel Pillows are machine washable and sized for most applications. One pair per package.

REF 6115 (Pastel Plaid)

REF 6115L (Pastel Plaid), Large
(Recommended for large or edematous foot/ankle)

REF 6116 (Navy)

Application Instructions for 6115, V6115, 6115L, 6116 and V6116:

1. Unfasten the hook and loop padded strap.
 2. Place the product on the foot so the seam is visible along the sole, not up the leg, and the padded strap can be secured across the ankle.
 3. Fasten the hook and loop padded strap for the desired fit.
- (NOTE: This product is not intended for use while walking.)

Foot Positioner

This product is designed for use with the 6115, V6115, 6115L, 6116 and V6116 Heel Pillows. The Foot Positioner provides additional support by holding the foot in dorsiflexion, minimizing foot drop. Side vents help keep the foot cool by promoting air circulation. This soft, lightweight, product is made of wipe clean material. Sold individually.

REF 6117I

Application instructions for the 6117I Foot Positioner (For use only with Posey 6115, 6115L, 6116, V6115 and V6116 Heel Pillows)

1. Position one of the above Posey Heel Pillows into the 6117I Foot Positioner. Thread the strap from the Heel Pillow through the slot. On the opposite inside panel secure the Heel Pillow to the loop fastener.

J.T. Posey Company

5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel: 800-447-6739 or 626-443-3143 • Fax: 800-767-3933 or 626-443-5014 • www.posey.com

© 1995 J.T. Posey Company. All rights reserved.




2. Place the combination Foot Positioner/Heel Pillow on the patient's foot, orientating the Foot Positioner so that the Heel Pillow attaching strap is on the outside of the patient's foot.
 3. Pull the strap across the patient's ankle securing both the Foot Positioner and Heel Pillow into place. Attach the strap to the hook fastener on the outside of the opposite side.
- (NOTE: This product is not intended for use while walking)

Washing/Cleaning Instructions:

Close hook and loop before washing.
6115, V6115, 6115L, 6116 and V6116



6117I - Wipe with mild soap and water or disinfectant.

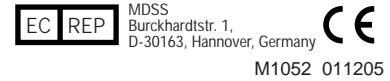

J.T. Posey Company 5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel: 800-447-6739 or 626-443-3143 • Fax: 800-767-3933 or 626-443-5014



Washing/Cleaning Instructions:
Close hook and loop before washing.
6113 & 6114



1. Unfasten the hook and loop padded straps.
 2. Place the product on the foot so that two straps can be secured around the calf and one strap secured across the toes.
 3. Fasten the hook and loop padded straps for the desired fit. The straps may be crossed for a small leg or to provide an extra snug fit.
- (NOTE: This product is not intended for use while walking.)

Application Instructions:

REF 6114 (Navy)

REF 6113 (Pastel Plaid)

The Posey Quilted Long Boot helps protect heels, ankles, and calves from skin irritation and painful pressure sores. Intended for patients who are seated or in bed. They are specially designed to minimize pressure on the heel by dispersing weight evenly along the calf. The protectors are filled with soft polyester fiber, covered with a cotton fabric, and have three padded straps with hook and loop closures to provide a snug, yet comfortable fit. Machine washable and one size fits most applications. One pair per package.

Posey® Quilted Long Boot

