



Posey® Wheelchair Slim Supports

Slim Supports for Wheelchair

An ideal solution that customizes a standard wheelchair for petite patients/residents. The slim support fits snugly inside the wheelchair seat to provide bilateral torso support. Made of 3/8" plywood covered with padded foam and wipe-clean vinyl for extra comfort. Adjustable straps for attachment. Fits desk arm or full length arm wheelchairs.

- Wipe-clean vinyl cover.
- One pair per package.
- Meets California #117 Flame Retardency Standards

Cat. #6322

Wheelchair Application Instructions:

1. Slim Supports are provided in pairs (right and left) and should be placed between the wheelchair side-plate and the patient. Place the zipper facing down, with the hard side of the support toward the side-plate, and the padded side toward the patient.
2. To secure the Slim Support, work the bottom front strap between the side-plate and the seat bottom until it clears the bottom of the chair. Pull the lower strap tightly, then connect it to the top strap on the outside of the side-plate. (See figure 1). To secure the straps, loop the bottom strap through the rings on the top strap (See figure 2) and pull tightly.
3. Wrap the rear strap around the rear post and secure it using the hook and loop closure.
4. Repeat the process for the other Slim Support.

Cleaning Instructions:

Wipe clean with disinfectant.



J.T. Posey Company 5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel: 800-447-6739 or 626-443-3143 • Fax: 800-767-3933 or 626-443-5014 • www.posey.com
 © 1995 J.T. Posey Company. All rights reserved. EU AUTH. PERSON: (MDD 93/42/EEC) MDSS GmbH, Burckhardtstr. 1, D-30163, Hannover, Germany



M1635 020101

J.T. Posey Company 5635 Peck Road • Arcadia, CA 91006-0020 USA • Tel: 800-447-6739 or 626-443-3143 • Fax: 800-767-3933 or 626-443-5014 • www.posey.com



Slim Supports for Gerchair

An ideal solution that customizes a gerchair for petite patients/residents. The slim support fits snugly inside the gerchair seat to provide bilateral torso support. Made of 3/8" plywood covered with padded foam and wipe-clean vinyl for extra comfort.

- Wipe-clean vinyl cover.
- One pair per package.
- Meets California #117 Flame Retardency Standards

Cat. #6323

Gerchair Application Instructions:

1. Slim Supports are provided in pairs (right and left) and should be placed between the side-plate and the patient. Place the zipper facing down, with the hard side of the support toward the chair armrest, and the padded side toward the patient. (See figure 1)
2. In figure 1, Slim Supports are shown in the horizontal position. This is suitable for shorter patients.
3. In figure 2, Slim Supports are shown in the vertical position. This is suitable for taller patients.

Cleaning Instructions:

Wipe clean with disinfectant.

Posey® Gerchair Slim Supports

