

DESCRIPTION OF PRODUCT: Activity lap top tray for wheelchairs or similar non-wheelchair applications.

Rx ONLY

8221



INDICATIONS FOR USE:

- Patients who have poor upper torso alignment and/or need a firm surface to rest their elbows on.
- Patients who require an anterior posture support.
- Patients who require a work area or eating area on their wheelchair or similar non-wheelchair applications.
- Product applications considered "self-release" or "assisted-release" must be specified by the ordering physician.



8220



CONTRAINDICATIONS:

- **DO NOT** use on a patient who is or becomes highly aggressive, combative, agitated, or suicidal.
- **STOP USE AT ONCE:** if the patient is at risk to slide forward or down in the device.
NOTE: A restraint with a pelvic piece will help to reduce the risk of sliding. See Posey Catalog.
- **DO NOT** use on a patient who is unwilling or unable to follow instructions, and is at risk of a fall or re-injury from self-release.

ADVERSE REACTIONS

- Severe emotional, psychological, or physical problems may occur: if the applied device is uncomfortable; or if it severely limits movement. If symptoms of these problems ever appear for any reason, get help from a qualified medical authority and find a less restrictive, product or intervention.

POSEY LAP TOP TRAYS

REF 8220 Clear Lap Top Tray
22"L x 20½" x ½" thick
(56 cm x 52 cm x 1.3 cm)

REF 8221 Padded Lap Top Tray
22"L x 20½" x ½" thick
(56 cm x 52 cm x 1.3 cm)

NOTE: These products are designed for use on full-length arm wheelchairs or other similarly equipped non-wheelchair applications.

APPLICATION INSTRUCTIONS:

1. Seat the patient in the chair with the hips resting firmly against the seat backrest.
2. Put the lap top tray on the armrest, so it doesn't interfere with breathing.
3. Secure the lap top tray to the arm rest using the provided "hook and loop" straps. There are two straps on each side. Either the front strap, back strap, or both may be used depending on the patient's assessment.
4. If necessary, use the strap around the back of the chair to prevent the tray from being pushed forward.

WARNING This device should ALWAYS be snug, but not interfere with breathing or circulation. After application, slide an open hand (flat) between the device and patient to ensure proper fit. ALWAYS monitor a patient to make sure the patient is not able to slide down, under the device, and fall off the chair seat (fig. 1). If a patient's body weight becomes suspended off the chair, chest compression and suffocation could result. Restraints with a pelvic piece may be necessary to reduce sliding down.



FIG. 1

ADDITIONAL SAFETY AND LAUNDERING INSTRUCTIONS ON OTHER SIDE



