

## MONITORING

**⚠️WARNING** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

## INSPECTION

**⚠️WARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

## SOFSTEP™ FOOTREST COVERS


Cushioned footrest covers help prevent cuts, bruises and skin tears and provide a skid-free surface. Easily slip on the footrest and remain securely in place. Covers fit all standard wheelchair foot plates.

**REF 6485** 5½" D x 6" W (14 cm x 15 cm)



## APPLICATION INSTRUCTIONS

1. Slip the SofStep™ over the wheelchair footplate (Fig. 1).
2. If there is resistance, moisten the inside of the SofStep™ with water and try again.

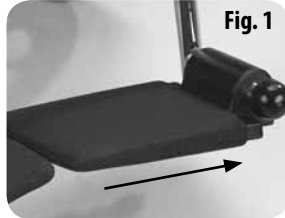

## WHEELCHAIR GOODIE BAG

Designed for carrying a patient's personal possessions, magazines, or cosmetics. Also may be used to carry a patient's chart. Fits right over push handles on virtually any wheelchair.

**REF 8211** 15½" W x 11" H (39 cm x 28 cm)



## APPLICATION INSTRUCTIONS

1. Slip handle over the push grips of the wheelchair (Fig. 2).
2. Adjust the straps, if necessary, to prevent bag from getting caught in the moving parts of the wheelchair.



## CLEANING INSTRUCTIONS:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



## DISPOSAL

**⚠️WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.