



# Posey® Full Side Rail Cover

Quality Products since 1937

BED SAFETY

## FULL SIDE RAIL COVER

The pad slips over the top of full siderail and goes across and under mattress to help prevent bedrail entanglement. A decubitus fur liner protects the patient who bangs against the hard metal bars. Rails may be raised or lowered without removing the siderail pad, and reinforced holes allow wrist restraints or belts to be secured to the bed frame. Machine washable. Synthetic fur lined.

**REF** 5702 82"L x 65"W

### APPLICATION INSTRUCTIONS:

1. Remove mattress from bed.
2. Slip Side Rail Covers over both side rails, fur side towards the patient.
3. Replace mattress.
4. Secure hook and loop fasteners at the ends of the siderails and to the movable part of the bed frame.

### WASHING INSTRUCTIONS



J.T. Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA  
Phone: 800-447-6739 • Fax: 800-767-3933 • Web: www.posey.com

**EC REP** MDSS  
Burckhardtstr. 1,  
D-30163, Hannover, Germany  
© 2006 J.T. Posey Company. All rights reserved.



M1626 072406



# Posey® Full Side Rail Cover

Quality Products since 1937

BED SAFETY

## FULL SIDE RAIL COVER

The pad slips over the top of full siderail and goes across and under mattress to help prevent bedrail entanglement. A decubitus fur liner protects the patient who bangs against the hard metal bars. Rails may be raised or lowered without removing the siderail pad, and reinforced holes allow wrist restraints or belts to be secured to the bed frame. Machine washable. Synthetic fur lined.

**REF** 5702 82"L x 65"W

### APPLICATION INSTRUCTIONS:

1. Remove mattress from bed.
2. Slip Side Rail Covers over both side rails, fur side towards the patient.
3. Replace mattress.
4. Secure hook and loop fasteners at the ends of the siderails and to the movable part of the bed frame.

### WASHING INSTRUCTIONS



J.T. Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA  
Phone: 800-447-6739 • Fax: 800-767-3933 • Web: www.posey.com

**EC REP** MDSS  
Burckhardtstr. 1,  
D-30163, Hannover, Germany  
© 2006 J.T. Posey Company. All rights reserved.



M1626 072406