

These gap protectors are designed for hospital beds that have split side rails with built-in patient controls. The firm padded protector fills in the gap between the rails, yet allows the patient access to the controls. Made of sturdy 1/4" (1 cm) plywood and covered in thick, foam-padded vinyl. Attaches with quick-release buckles. Vinyl cover wipes clean. One pair per package. Meets CA #117, Section E Flame Retardancy Standards.

REF 5706 Gap Protector, 34"L x 12"H x 3/4"T
(86 cm x 30 cm x 2 cm)

Monitoring

⚠ WARNING Posey Gap Protectors will not prevent a determined patient from exiting the bed, going over or attempting to go between or through side rail gaps. **DO NOT** articulate bed if product is applied. **ALWAYS** follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

Application Instructions:

1. Inspect product prior to use. **DO NOT** use soiled or damaged products.
2. Orient the Posey Gap Protector inside and between the side rails so the gap between the rails is closed. Be sure the control panel is accessible to the patient.
3. Wrap the nylon straps over the top of the side rails and attach them to the outside of the protector using the quick-release buckles.
4. Adjust straps for a snug fit.
5. Protector must be removed before lowering the side rails.
6. Check for proper fit. Modified beds may not be compatible with Posey Gap Protectors.



Cleaning Instructions:

Wipe clean with soap and water or disinfectant. **DO NOT** use cleaners that contain Phenol or Benzyl.



Disposal

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

Storage And Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.