

The contoured, Orthotic Drop Seat has an adjustable drop up to 3½" (9 cm) and is designed to help enable easier chair walking. Removable wipe-clean padded vinyl covers provide comfort and easy maintenance. 250 lb. (113 kgs.) maximum weight capacity.



REF 6331 Orthotic Drop Seat - fits many 18" and 20" (46 cm and 51 cm) wheelchairs

MONITORING

AWARNING Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation

INSPECTION

AWARNING Inspect before each use: check for broken stitches or parts; torn, cut or frayed material; or buckles that do not hold securely. DO NOT use soiled or damaged products.

INSTALLATION INSTRUCTIONS

(Tools required: #2 Phillips Screwdriver)

1. Remove the vinyl wheelchair seat by removing the screws from the horizontal seat rail.
2. Position the wheelchair on its back, so it is resting on the hand grips and wheels. You should now be looking at the bottom of the seat section.
3. Remove the padded vinyl cover from the product.
4. Position the Orthotic Drop Seat so the pommel is facing up and forward.
5. Thread the non-buckle end of the strap through the left front slot on the under side of the seat (**A**). Then thread UNDER AND OVER the upper most horizontal bar of the wheelchair (**B**) and pass the strap back through the same front slot (**C**) (Fig. 1).



May be configured as a wedge to help prevent forward sliding.

NOTE:

When properly installed, all straps should be under the seating surface except those that come out the slots and encircle the uppermost horizontal bar of the wheelchair.

AWARNING ALWAYS test the strength and security of the product after installation. DO NOT seat a person until you have made sure that the product is applied securely and will hold the weight of the patient. When installed properly, this product will withstand weights equal to that of the wheelchair for which it is intended.

CLEANING INSTRUCTIONS:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.

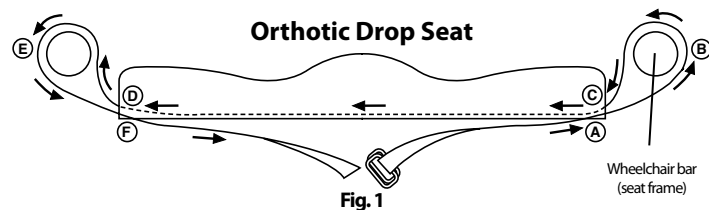


Fig. 1

6. Continue threading the strap under the seat and through the opposing front slot on the left side of the seat (**D**).
7. Thread OVER AND UNDER the (**E**) uppermost horizontal bar of the wheelchair and pass it back through the left side slot (**F**).
8. Feed the strap through the two square loops and back through the first one (Fig. 2).
9. Repeat steps 5 - 8 on the rear slots on the seat.
10. Replace padded cover.



Fig. 2

DISPOSAL

AWARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.