

MONITORING

⚠️WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.



INSPECTION

⚠️WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

⚠️CAUTION DO NOT allow the patient to walk or ambulate while wearing these products.

VENTILATED HEEL PROTECTOR

Ventilated heel allows the foot to "breathe." Side vent holes reduce heat buildup. Ideal for patients with edema. Hook and loop fasteners ensure a quick and easy application. One size fits all.

REF 6121 6"L x 6"H (15 cm x 15 cm)

APPLICATION INSTRUCTIONS

Position patient's foot into the Economy Heel Protector. Attach loop on straps to hook patch on boot. Straps may be trimmed if necessary.



6121

SURE STAY HEEL PROTECTOR

Difficult for even the most active patient to kick off; the unique design also helps prevent the heel protector from riding up on the patient's leg. One size fits all.

REF 6125 8"L x 7"H (20 cm x 18 cm)

APPLICATION INSTRUCTIONS

Place the patient's foot inside the heel protector. Attach loop on straps to hook patch on boot. Straps may be trimmed if necessary.



6125

WASHING/CLEANING INSTRUCTIONS: ALL ITEMS

Remove any plastic shells before laundering



Plastic Shells--

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



ECONOMY HEEL PROTECTOR

Ventilated heel allows the foot to "breathe" Model 6110 has a slick plastic shell that slides easily over the sheets. Easy to apply hook and loop fastener. One size fits all.

REF 6110 with plastic shell, 5"L x 5"H (13 cm x 13 cm)

REF 6120 without plastic shell, 5"L x 5"H (13 cm x 13 cm)

APPLICATION INSTRUCTIONS

Place on patient's foot with plastic shell on the outside of the foot. Draw ankle strap through the plastic shell and bring back on itself to fasten.



6110

LONG BOOT

A mid-calf boot for bedridden patients with sensitive skin. Hook and loop fasteners ensure a snug, easy fit. One size fits all.

REF 6112 14"L x 9"H (36 cm x 23 cm)

APPLICATION INSTRUCTIONS

Position the patient's foot into the Long Boot. Attach loop on straps to hook patch on boot. Straps may be trimmed if necessary.



6112

COMFORT SHOE

The Comfort Shoe is designed to provide full foot protection and help prevent friction burns. Form fitting with hook and loop closures. One size fits all.

REF 6415 Closed Toe, 12"L x 10"H (30 cm x 25 cm)

REF 6416 Open Toe, 12"L x 10"H (30 cm x 25 cm)

APPLICATION INSTRUCTIONS

Position the patient's foot into the comfort shoe. Attach loop on straps to hook patch on shoe. Note: Hook patches face outward from the foot. Straps may be trimmed, if necessary.



6415

DISPOSAL

⚠️WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.