

PILLOW PAL

The Posey Pillow Pal is a versatile product. When placed between the legs, the Pillow Pal helps alleviate back pain by maintaining spine and hip alignment. When placed behind the head, the Pillow Pal helps facilitate head and neck alignment. The pillowcase helps prevent contamination of the Pillow Pal.



FEATURES:

- Measures 8" L x 10½" W (20 cm x 27 cm).
- Polyurethane foam construction.
- Unique slit design helps provide flexibility.

REF 6319 Pillow Pal, complete set with pillowcase

MONITORING

⚠WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

INSPECTION

⚠WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS

LEG APPLICATION:


1. Place the flat section of the Pillow Pal between the legs, slightly above the knees (Fig. 1). Adjust the position of Pillow Pal as appropriate for maximum comfort.

HEAD APPLICATION:

1. Position the flat section of the Pillow Pal under the head to aid in proper head and neck alignment. The Pillow Pal can be placed behind the head either lengthwise (Fig. 2) or width wise (Fig. 3) depending on the user's preference. Adjust position of Pillow Pal as appropriate.

LAUNDERING INSTRUCTIONS:

Posey Pillow Pal Pillowcase **only**:

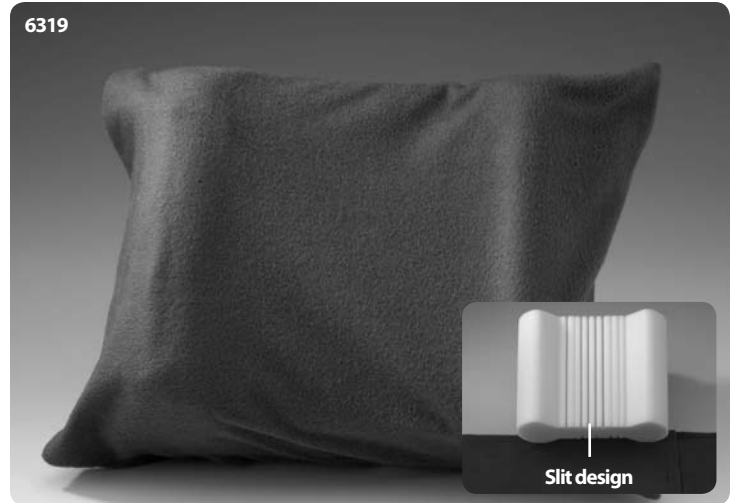

DISPOSAL

⚠WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.