

INDICATION:

Wedge Cushions are designed to help reduce forward sliding, without the need for additional belts or restraints.



PRODUCT FEATURES:

- Helps offer exceptional seating comfort by positioning the patient's hips in a forward angle.
- Gel foam inner construction helps redistribute contact pressures away from concentrated areas.
- Vinyl cover is waterproof to protect the inner foam core.
- Provides a stable, comfortable seating surface.
- Zippered outer cover can be quickly removed for cleaning.
- Meets CA #117 flame retardancy standards.

MONITORING


WARNING Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

INSPECTION

WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products. Keep sharp objects away from cushion as they may puncture the gel bladders.

APPLICATION INSTRUCTIONS

1. Make sure the cushion dimensions match those of the wheelchair.
2. Place cushion on the wheelchair seating surface.
3. Align the front edge of the cushion with the front edge of the wheelchair seat.

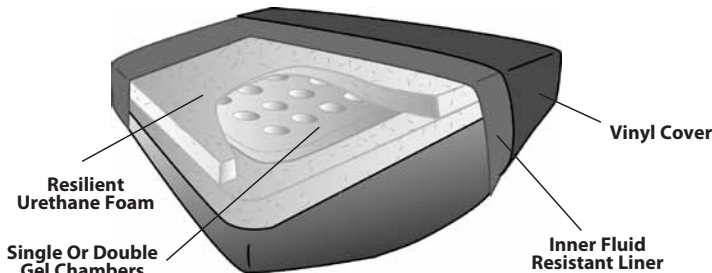

6620C shown with Convex bottom



CLEANING INSTRUCTIONS:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



DISPOSAL

WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials. Store cushion horizontally.

CAT. #	WEIGHT CAPACITY	BOTTOM	DIMENSIONS W x L x H	CUSHION WEIGHT	COVER COLOR
6610C	Standard/200 lbs. / 91 kgs.	Convex	16" x 16" x 4" - 2" 41 cm x 41 cm x 10 cm - 5 cm	6 lbs. / 2.7 kgs.	Dark blue
6610F	Standard/200 lbs. / 91 kgs.	Flat	16" x 16" x 4" - 2" 41 cm x 41 cm x 10 cm - 5 cm	6 lbs. / 2.7 kgs.	Dark blue
6620C	Standard/200 lbs. / 91 kgs.	Convex	18" x 16" x 4" - 2" 46 cm x 41 cm x 10 cm - 5 cm	6.5 lbs. / 2.9 kgs.	Dark blue
6620F	Standard/200 lbs. / 91 kgs.	Flat	18" x 16" x 4" - 2" 46 cm x 41 cm x 10 cm - 5 cm	6.5 lbs. / 2.9 kgs.	Dark blue
6630F	Bariatric/300 lbs. / 136 kgs.	Flat	20" x 18" x 5" - 3" 51 cm x 46 cm x 13 cm - 8 cm	14 lbs. / 6.4 kgs.	Dark blue

A full seating assessment is recommended to determine the appropriate size and configuration for your patient/resident.

WARRANTY:

All Posey Deluxe Wedge Gel Foam Cushions are warranted to perform as indicated. The Posey Company will, at their option, repair or replace, except for vandalism or other abuse, any cushion for a period of one (1) year from date of purchase.