

## INDICATION:

Posey Economy Gel Foam Cushions are designed to redistribute pressure away from high pressure contact points, to help improve patient comfort. Ideal for use in standard wheelchairs or geri-chairs.



## PRODUCT FEATURES:

- Unique gel foam inner bladder construction.
- Waterproof vinyl cover.
- Meets CA #117 flame retardancy standards.

## MONITORING


**WARNING** Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

## INSPECTION

**WARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products. Keep sharp objects away from cushion as they may puncture the gel bladders.

## APPLICATION INSTRUCTIONS

1. Make sure the cushion dimensions match those of the wheelchair.
2. Place cushion on the wheelchair seating surface.
3. Align the front edge of the cushion with the front edge of the wheelchair seat.

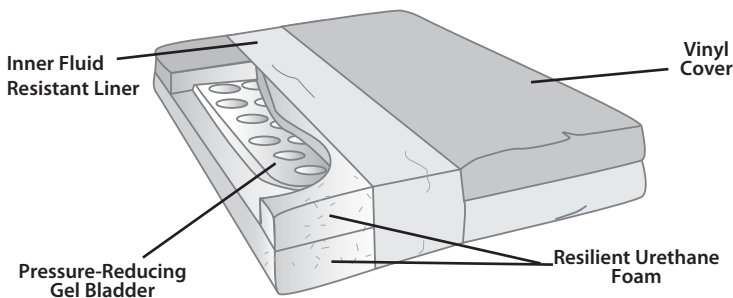

## CLEANING INSTRUCTIONS:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



## DISPOSAL

**WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials. Store cushion horizontally.

CUSHION	DIMENSIONS W x L x H	CUSHION WEIGHT	COVER COLOR	WEIGHT CAPACITY
<b>7310F</b>	16" x 16" x 2" 41 cm x 41 cm x 5 cm	6 lbs. / 2.7 kgs.	Dark Blue	200 lbs. / 91 kgs.
<b>7320F</b>	18 x 16 x 2 46 cm x 41 cm x 5 cm	6.5 lbs. / 2.9 kgs.	Dark Blue	200 lbs. / 91 kgs.
<b>7320FT</b>	18 x 16 x 2 46 cm x 41 cm x 5 cm	6.5 lbs. / 2.9 kgs.	Dark Blue/Maroon	200 lbs. / 91 kgs.
<b>7321F</b>	18 x 16 x 2 46 cm x 41 cm x 5 cm	6.5 lbs. / 2.9 kgs.	Light Blue	200 lbs. / 91 kgs.
<b>7330F</b>	20 x 18 x 2 51 cm x 46 cm x 5 cm	6.5 lbs. / 2.9 kgs.	Dark Blue	200 lbs. / 91 kgs.

## WARRANTY:

All Posey Economy Gel Foam Cushions are warranted to perform as indicated. The Posey Company will, at their option, repair or replace, except for vandalism or other abuse, any cushion for a period of one (1) year from date of purchase.