

INDICATION:

Posey Deluxe Gel Foam Cushions are designed to redistribute pressure away from high pressure contact points, to help improve patient comfort and help control against sliding. Ideal for use in standard or bariatric wheelchairs, or in geri-chairs.



PRODUCT FEATURES:

- A full seating assessment is recommended to determine the appropriate size and configuration for your patient/resident.
- Zippered outer cover can be quickly removed for cleaning.
- Meets CA #117 flame retardancy standards.
- 20", 22", and 24" (51 cm, 56 cm and 61 cm) wide cushions feature dual gel chambers.

MONITORING

⚠WARNING Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

INSPECTION

⚠WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products. Keep sharp objects away from cushion as they may puncture the gel bladders.

APPLICATION INSTRUCTIONS

1. Make sure the cushion dimensions match those of the wheelchair.
2. Place cushion on the wheelchair seating surface.
3. Align the front edge of the cushion with the front edge of the wheelchair seat.



CLEANING INSTRUCTIONS:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



DISPOSAL

⚠WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials. Store cushion horizontally.

WARRANTY:

All Posey Deluxe Gel Foam Cushions are warranted to perform as indicated. The Posey Company will, at their option, repair or replace, except for vandalism or other abuse, any Cushion for a period of one (1) year from date of purchase.

CUSHION	DIMENSIONS W x L x H	BOTTOM WEIGHT	COVER DESIGN	CUSHION WEIGHT	COLOR
7210F	16" x 16" x 2" (41 cm x 41 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6 lbs. / 2.7 kgs.	Dk. Blue
7210FT	16" x 16" x 2" (41 cm x 41 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6 lbs. / 2.7 kgs.	Dk. Blue/Maroon
7220F	18" x 16" x 2" (46 cm x 41 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6.5 lbs. / 3 kgs.	Dk. Blue
7220FT	18" x 16" x 2" (46 cm x 41 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6.5 lbs. / 3 kgs.	Dk. Blue/Maroon
7221F	18" x 16" x 2" (46 cm x 41 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6.5 lbs. / 3 kgs.	Pink/Lt. Blue
7222F**	18" x 16" x 2" (46 cm x 41 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6.5 lbs. / 3 kgs.	Red/Blue
7223F/7224F*	18" x 16" x 2" (46 cm x 41 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6.5 lbs. / 3 kgs.	Dk. Blue
7225C	18" x 16" x 3" (46 cm x 41 cm x 8 cm)	200 lbs. / 91 kgs.	Convex	6.5 lbs. / 3 kgs.	Dk. Blue
7226F	18" x 18" x 2" (46 cm x 46 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6.5 lbs. / 3 kgs.	Dk. Blue
7226FT	18" x 18" x 2" (46 cm x 46 cm x 5 cm)	200 lbs. / 91 kgs.	Flat	6.5 lbs. / 3 kgs.	Dk. Blue/Maroon
7230F	20" x 18" x 3" (51 cm x 46 cm x 8 cm)	300 lbs. / 136 kgs.	Flat	14 lbs. / 6.4 kgs.	Dk. Blue
7230FT	20" x 16" x 3" (51 cm x 41 cm x 8 cm)	300 lbs. / 136 kgs.	Flat	14 lbs. / 6.4 kgs.	Dk. Blue/Maroon
7240F	22" x 18" x 3" (56 cm x 46 cm x 8 cm)	400 lbs. / 181 kgs.	Flat	14.5 lbs. / 6.6 kgs.	Dk. Blue
7240FT	22" x 18" x 3" (56 cm x 46 cm x 8 cm)	400 lbs. / 181 kgs.	Flat	14.5 lbs. / 6.6 kgs.	Dk. Blue/Maroon
7250F	24" x 18" x 3" (61 cm x 46 cm x 8 cm)	400 lbs. / 181 kgs.	Flat	14.5 lbs. / 6.6 kgs.	Dk. Blue
7250FT	24" x 18" x 3" (61 cm x 46 cm x 8 cm)	400 lbs. / 181 kgs.	Flat	14.5 lbs. / 6.6 kgs.	Dk. Blue/Maroon

* Cushion cover features a pouch to hold a chair alarm sensor. Cat. #7224 comes complete with chair pad sensor.

** Cushion cover features one side vinyl and one side brushed polyester.

