



# Posey® Molding Foam Cushion and Foam Cushion with Coccyx Cutout

SEATING AND POSITIONING

## MOLDING FOAM CUSHION

### INDICATIONS:

Patients at risk for pressure ulcers and/or requiring posture support and/or inclined to slide forward.



### PATIENT BENEFITS:

- Constructed from 1½" (4 cm) thick viscoelastic foam ("memory foam") laminated to a firm urethane base.
- Provides stability and pressure distribution, helping to reduce the risk of pressure ulcer development.
- Unique rounded front edge design reduces pressure under the knee and thighs.
- Convex shaped bottom helps compensate the "hammocking" effect of sling wheelchair seats.
- Zippered outer cover removes quickly and easily for cleaning.



## FOAM CUSHION WITH COCCYX CUTOUT

### INDICATIONS:

For patients at risk of pressure ulcers in the coccyx area.

### PATIENT BENEFITS:

- Urethane foam inner construction with Posey's Sure-Grip vinyl covers to provide a smooth seating surface.
- Cutout in the back helps provide offloading of the coccyx.



CAT. #	CUSHION	DIMENSIONS W" x L" x H"	BOTTOM	WEIGHT
6910F	Foam Cushion w/cutout	16 x 16 x 3 (41 cm x 41 cm x 8 cm)	Flat	1 lb. (0.45 kgs.)
6920F	Foam Cushion w/cutout	18 x 16 x 3 (46 cm x 41 cm x 8 cm)	Flat	1 lb. (0.45 kgs.)
7024C	Molding Foam Cushion	18 x 16 x 3 (46 cm x 41 cm x 8 cm)	Convex	3 lbs. (1.4 kgs.)

All Cushions meet CA #117 flame retardancy standards.  
Maximum weight capacity - 250 lbs. (114 kgs.).

## MONITORING

**⚠WARNING** Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

## INSPECTION

**⚠WARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

## APPLICATION INSTRUCTIONS

1. Make sure the cushion dimensions match those of the wheelchair.
2. Place cushion on the wheelchair seating surface.
3. Align the front edge of the cushion with the front edge of the wheelchair seat.

## CLEANING INSTRUCTIONS - COVER ONLY

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



## DISPOSAL

**⚠WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

## WARRANTY:

All Posey Foam Cushions are warranted to perform as indicated. The Posey Company will, at their option, repair or replace, except for vandalism or other abuse, any Cushion for a period of one (1) year from date of purchase.

## OPTIONAL ACCESSORY:

The Posey Wheelchair Cushion Incontinence Pad is made from a durable quilted polyester laminate. Made of highly absorbent material, the pad is designed to help protect against urine spills. Non-latex elastic straps at all four corners secure the pad to the cushion. Will absorb and hold up to 300cc, or 10 oz. of fluid. Fits cushions up to 20"W x 18"D x 2"H (51 cm x 46 cm x 5 cm).

**REF 6303** Incontinence Pad (Each)  
20"W x 18"D (51 cm x 46 cm)



Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • www.posey.com  
Phone: 1.800.447.6739 or 1.626.443.3143 • Fax: 1.800.767.3933 or 1.626.443.5014