

## SEMI-RIGID WHEELCHAIR FOUNDATION

The Posey Semi-Rigid Wheelchair Foundation is designed to help counteract the natural “hammocking” effect of soft vinyl sling seats found in standard wheelchairs. Can be used to help provide a firm foundation for any flat cushion.



### PATIENT BENEFITS:

- Solid-foam core construction conforms to the wheelchair hammock seat while providing a flat surface to help support a wheelchair cushion.
- Helps provide a solid surface, thus eliminating uneven weight distribution on the ischial tuberosities.
- Features a vinyl outer cover, which helps provide a firm seating surface that helps reduce cushion sliding.
- Sits flat and low in the chair.
- Meets CA #117 flame retardancy standards.

### MONITORING

**⚠WARNING** Be sure to follow your facility’s policies and guidelines for frequency of patient monitoring. This product must be used in conjunction with a wheelchair cushion.

### INSPECTION

**⚠WARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

### APPLICATION INSTRUCTIONS

1. Make sure the foundation cushion dimensions match those of the wheelchair.
2. Place foundation cushion on the wheelchair seating surface.
3. Align the front edge of the foundation with the front edge of the wheelchair seat.

### CLEANING INSTRUCTIONS:

Models with vinyl covers only.

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



### DISPOSAL

**⚠WARNING** Properly dispose of the product per facility’s policy for BIOHAZARDOUS materials.

### STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials. Store cushion horizontally.

### WARRANTY:

All Posey Semi-Rigid Wheelchair Foundations are warranted to perform as indicated. The Posey Company will, at their option, repair or replace, except for vandalism or other abuse, any cushion for a period of one (1) year from date of purchase.

SEMI-RIGID FOUNDATION	BOTTOM	DIMENSIONS W x L x H	WEIGHT	COVER COLOR	COVER STYLE	WEIGHT CAPACITY
7110C	Convex	16" x 16" x 1½" (41 cm x 41 cm x 4 cm)	1 lb. (0.45 kgs.)	Dark Blue	Vinyl	200 lbs. (91 kgs.)
7120C	Convex	18" x 16" x 1½" (46 cm x 41 cm x 4 cm)	1 lb. (0.45 kgs.)	Dark Blue	Vinyl	200 lbs. (91 kgs.)
7121C	Convex	18" x 16" x 1½" (46 cm x 41 cm x 4 cm)	1 lb. (0.45 kgs.)	No Cover	No Cover	200 lbs. (91 kgs.)
7130C	Convex	20" x 16" x 1½" (51 cm x 41 cm x 4 cm)	1 lb. (0.45 kgs.)	Dark Blue	Vinyl	200 lbs. (91 kgs.)