


INDICATION:

Patients at low risk for pressure sores and/or requiring posture support. 

ECONOMY FOAM CUSHIONS

Firm foam foundation helps redistribute pressure for the cost-conscious buyer. Reversible model with two-tone cover to remind staff to reposition the patient/resident at regular intervals. Meets CA #117 flame retardancy standards.

MONITORING

AWARNING Always follow your facility's policies and procedures for patient assessment, monitoring and rehabilitation.

INSPECTION

AWARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS

1. Make sure the cushion dimensions match those of the wheelchair.
2. Place cushion on the wheelchair seating surface.
3. Align the front edge of the cushion with the front edge of the wheelchair seat.

CLEANING INSTRUCTIONS:

- Wipe Clean with mild detergent. OSHA approved intermediate level disinfectants can be used per manufacturer instructions. DO NOT use phenol and benzyl based disinfectants.
- After cleaning, products MUST be rinsed with water to remove any residual chemicals.
- Make sure products are completely dry before use.



DISPOSAL

AWARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



WARRANTY:

Posey Economy Foam Cushions are warranted to perform as indicated. The Posey Company will, at their option, repair or replace, except for vandalism or other abuse, any Cushion for a period of one (1) year from date of purchase.

CUSHION	DIMENSIONS W X D X H	WEIGHT	BOTTOM DESIGN	COLOR
7025FT	18" x 16" x 2 46 cm x 41 cm x 5 cm	1 lb. 0.45 kg	Flat	Dark Blue/Maroon Vinyl
7025F	18" x 16" x 2" 46 cm x 41 cm x 5 cm	1 lb. 0.45 kg	Flat	Pink/Blue Vinyl