

The Posey Torso Cushion is designed to position the patient comfortably, and support the patient's head and torso when the head of a hospital bed is down. The Posey Torso Cushion may also be used as an alternative for certain at-risk patients who may be at risk of serious injury or death from entrapment in the "gaps" or "pockets" at the head of the Posey StaySafe[™] Bed when the head of the bed is raised. Materials used meet California #117, Section E Flame Retardancy Standards.

Posey recommends that you leave the head of the bed down (flat) and use the Posey Torso Cushion in the Posey StaySafe Bed if you need to elevate the head or torso of an at-risk patient (for example, for watching TV or if required by the physician's orders or the care plan).

REF 8025 Posey Torso Cushion

At-Risk Patients Include

- Patients who, because of mental or physical disability, cannot reposition or remove him or herself from a confined area in the bed.
- Any other patient identified by the physician, the care plan, or Interdisciplinary Healthcare Team (IDHT) to be at risk of entrapment in the gaps created when the head of the Posey StaySafe Bed is up.

Instructions for Use:

1. Lower the head of the mattress to a flat position.
2. Place the Posey Torso Cushion on the bed at the head area.
3. Based on the patient's needs, the cushion can be positioned at a 30 degree angle, or turned over to create a 60 degree angle.

⚠ WARNING

The Posey Torso Cushion is designed for use when the head of the bed is down (flat). If the head of the bed is raised, creating "gaps" or "pockets" between the mattress and side rail or mattress and side panel of the Posey StaySafe Bed, use Posey Filler Cushions (Cat. No. 8021) to help reduce the risk of entrapment.

REMOVE the Posey Torso Cushion when not in use.

Laundering Instructions:



Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

Disposal

⚠ WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

