

## FOAM FOOT ELEVATOR/FOAM FOOT STABILIZER

The Posey Foam Foot Elevator and Posey Foam Foot Stabilizer help protect and treat heel ulcers and foot drop. Both products feature an adjustable strap that helps keep the foot upright in a neutral position. The strap secures with hook and loop closures. Both products are constructed with dense polyurethane foam and are intended for single patient use. One per package.



**REF 6128** Foam Foot Elevator, 6”L x 8” H (15 cm x 20 cm)

**REF 6129** Foam Foot Stabilizer, 12”L x 8”H (30 cm x 20 cm)

### MONITORING

**WARNING** Be sure to follow your facility’s policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

### INSPECTION

**WARNING** Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

### APPLICATION INSTRUCTIONS:

1. Release the adjustable foot strap.
2. Unfasten the hook and loop closures on the top to “open” the elevator/stabilizer.
3. Wrap the elevator/stabilizer around the patient’s ankle; secure with hook and loop closures.
4. Secure foot in neutral position with adjustable foot drop strap.

**NOTE:** To minimize pressure on the ankle and hyperextension of the knee, place a pillow under the patient’s knee and calf.

### DISPOSAL

**WARNING** Properly dispose of the product per facility’s policy for BIOHAZARDOUS materials.

### STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

