

**DESCRIPTION OF PRODUCT:** Posey SkinSleeves® protectors are versatile, non-compression knitted skin coverings suitable for a variety of applications. They help provide a protective covering against tears and abrasions for thin, fragile skin tissue. The soft cotton/nylon/spandex 4-way stretch yarn is not made with natural rubber latex and adapts to a wide variety of individuals.

### Fitting Guidelines

SkinSleeves protectors are intended to fit as a comfort product. Actual fit will depend on individual anthropometric measurements and patient comfort level. Posey regular SkinSleeves protectors will stretch up to 100% in size (combination of both width and length). Padded sizes have limited stretchability.

**CAUTION** The higher the degree of stretching, the tighter the fit and the higher amount of compression on the patient's skin and muscle.

### Arm Application Instructions

1. Beginning with upper arm opening, roll inside edge of the SkinSleeves protector outwards; continue to roll until you reach the thumb opening.
2. Slip the arm opening of the SkinSleeves protector over the patient's hand, fitting the patient's thumb through the thumb hole.
3. Unroll the SkinSleeves protector back up over the arm. Adjust to desired placement.
4. If desired, the patient's thumb can be removed from the thumb opening and the SkinSleeves protector "rolled up" on the wrist to uncover the palm.

### Leg Application Instructions

1. Beginning with upper leg opening, roll inside edge of SkinSleeves protector outwards until you get within 3" of the toe opening.
2. Slip the upper leg opening of the SkinSleeves protector over the patient's foot, pulling it up until the heel "crook" of the SkinSleeves protector fits snugly over the patient's heel.
3. Unroll the SkinSleeves protector up the leg. Adjust to desired placement.

### Monitoring

**WARNING** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

### Inspection

**WARNING** Inspect skin frequently for tears, lacerations, avulsions, abrasions, infection or other damage and determine appropriate treatment. Change product with new or laundered product in accordance with facility policy.


### To Launder

Machine wash and dry low temperature.  
For best results, hang dry.



### Disposal

**WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

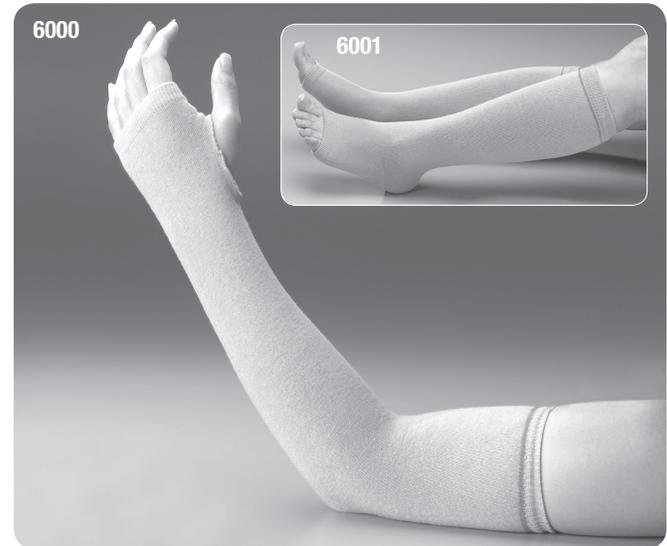

### Posey SkinSleeves Protectors

Item Number	Size/Color band	Length	Recommended arm circumference for a comfort fit
REF 6000S	Arm - Small/Red	15½" (39½ cm)	8½" (21½ cm)
REF 6003S	Arm - Small/Red	15½" (39½ cm)	8½" (21½ cm)
REF 6000	Arm - Medium/Green	16½" (42 cm)	11" (28 cm)
REF 6000B	Arm - Medium/Green	16½" (42 cm)	11" (28 cm)
REF 6002	Arm - Medium/Green	16½" (42 cm)	11" (28 cm)
REF 6003	Arm - Medium/Green	16½" (42 cm)	11" (28 cm)
REF 6000L	Arm - Large/Yellow	18" (46 cm)	11" (28 cm)
REF 6000XL	Arm - X-Large/Blue	20" (51 cm)	13" (33 cm)
REF 6001	Leg - Medium/Green	13½" (42 cm)	11" (28 cm)
REF 6004	Leg - Medium/Green	13½" (42 cm)	11" (28 cm)
REF 6005	Leg - Medium/Green	13½" (42 cm)	11" (28 cm)

### Storage and Handling

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.