



Posey®

Heel and Elbow Protectors

Quality Products since 1937

WOUND PREVENTION

Posey Knitted Heel and Elbow Protectors Offer Comfort and Protection!

INDICATIONS:

Foam Pad - Low risk patients requiring comfort, warmth and/or protection against abrasions from contact surfaces.

Gel Pad - Low risk patients requiring pressure relief from contact surfaces or protection when there is a concern for falls.



Posey Knitted Heel and Elbow Protectors offer comfortable sock-like protective coverings made from a soft stretchable yarn. The yarn is a unique blend of acrylic/nylon/spandex. Features woven-in size markings. Machine washable and Latex-free. One pair per package.

The Posey #6224 Protectors are intended for use in conjunction with facility dermal ulcer therapy and nursing protocols.

- REF 6224S Foam Pad Small
- REF 6224M Foam Pad Medium
- REF 6224L Foam Pad Large
- REF 6224XL Foam Pad X-Large

- REF 6224GS Gel Pad Small
- REF 6224GM Gel Pad Medium
- REF 6224GL Gel Pad Large
- REF 6224GXL Gel Pad X-Large



6224 - Foam pad inner liner



6224G - "Gel" pad inner Liner

FREE TRIAL SAMPLES: CALL 1.800.447.6739

SIZING AND ORDERING INFORMATION

SIZE	FITS LIMB CIRCUMFERENCE UP TO:	FOAM PAD	GEL PAD
Small	8½" (21½ cm)	6224S	6224GS
Medium	11" (28 cm)	6224M	6224GM
Large	13½" (34 cm)	6224L	6224GL
X-Large	16" (40½ cm)	6224XL	6224GXL

FITTING GUIDELINES

Knitted Heel/Elbow Protectors are intended to fit as a comfort product. Actual fit will depend on individual anthropometric measurements and patient comfort level. These protectors will stretch up to 30%.

MONITORING

⚠️WARNING Be sure to follow your facility's policies and guidelines for frequency of patient monitoring. Posey recommends that this product be removed at least every two (2) hours to check for skin integrity, proper circulation and range of motion.

INSPECTION

⚠️WARNING Inspect before each use: check for broken stitches; or torn, cut or frayed material. DO NOT use soiled or damaged products.

APPLICATION INSTRUCTIONS

1. Make sure the skin area to be covered by the Heel and Elbow Protector is free of any open wounds or pressure ulcers.
2. Select the appropriate size for the patient's heel or elbow.
3. Slip the Heel and Elbow Protector over the foot or arm and position the internal contoured pad over the area to be cushioned.
4. The Heel and Elbow Protector should fit snugly on the limb. If it appears to be too tight or loose, choose the next most appropriate size to ensure patient comfort.

⚠️WARNING Inspect skin frequently for tears, lacerations, avulsions, abrasions, infections or other damage and determine appropriate treatment. Change product with new or laundered product in accordance with facility policy.

To help prevent patient slips or falls, patient should not be allowed to walk when wearing the Posey #6224 and #6224G Protectors on their heels.

LAUNDERING INSTRUCTIONS



WASH HOT
25 MIN.



BLEACH AS DIRECTED
ON CONTAINER



DRY ON
LOW

DISPOSAL

⚠️WARNING Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.



MDSS GmbH
Schiffgraben 41
D-30175 Hannover, Germany

