

Body Holder 1731 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS: FOR CHAIR USE: 1. Position the patient as far back in the seat as possible, with the buttocks against the back of the chair.			
2. Wrap the belt around the hips, flannel side toward the patient. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.			
3. Bring the ends of the connecting straps down at a 45-degree angle between the seat and wheelchair sides. Criss-cross the straps behind the chair and draw them around the opposite side kick spurs.			
4. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the chair is adjusted.			
APPLICATION INSTRUCTIONS: BED (Increased Movement) 1. Wrap the body holder around the patient's waist with the belt slot in the rear, flannel side toward the patient. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.			

Rx ONLY

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com



Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
2. Bring the straps through the slot, and secure each to a movable part of the bed frame, out of the patient's reach, using quick-release ties.			
3. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the bed is adjusted.			
APPLICATION INSTRUCTIONS:			
BED (Decreased Movement)			
1. Wrap the body holder around the patient's waist with the belt slot in the front, flannel side toward the patient. The belt must be snug, but not interfere with breathing. To check for proper fit, slide an open hand (flat) between the belt and the patient.			
2. Bring the straps through the slot, and secure each to a movable part of the bed frame, out of the patient's reach, using quick-release ties.			
3. Check that the straps are secure and will not change position, loosen, or tighten if the patient pulls on them, or if the bed is adjusted.			