

Heel Pillows 6115, 6115L, 6116 CHECKLIST



Facility		Contact	
Facility Name		Name	
Street Address		Title	
City/State/Zip		Nursing Unit	
Instructor		Date	
Signature			

Description	Competency Level	Date	Comments
Competency Level Definition - (1) Needs remediation and re-evaluation, (2) Performs skill with accuracy under supervision, (3) Performs skill with accuracy independently			
APPLICATION INSTRUCTIONS: 1. Select Heel Pillow in correct size/style: 6115 Pastel Plaid, 10"L x 9"H (26 cm x 23 cm) 6115L Pastel Plaid, Large, 11½"L x 11"H (29 cm x 28 cm) 6116 Navy, 10"L x 9"H (26 cm x 23 cm)			
2. Unfasten the hook and loop padded strap.			
3. Place the product on the foot so the seam is visible along the sole, not up the leg, and the padded strap can be secured across the ankle.			
4. Fasten the hook and loop padded strap for the desired fit.			
5. Follow your Hospital's policy and procedures for assessing patient while the product is in use.			

Before use, make sure to read the instructions accompanying the product. A copy of the instruction sheet is included with the product and can also be downloaded at www.posey.com.

5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 • Fax: 1.800.767.3933 • www.posey.com