

How to Tie Posey Quick-Release Ties



Fig. 1



Fig. 2



Fig. 3



Fig. 4



Fig. 5

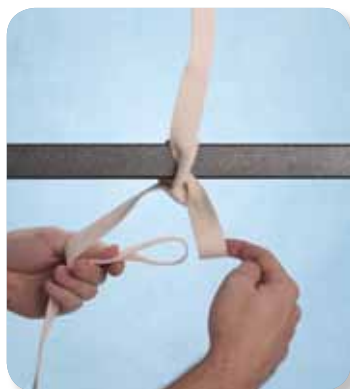


Fig. 6



Fig. 7



Fig. 8



Fig. 9

Properly tied
Quick-Release
knots will release
with one pull of
the loose end.

When using a restrictive product that must be tied, always secure the product using quick-release ties. When using a restraint in bed, always attach the straps or ties to the movable part of the bed frame at a juncture of the frame where they will not slide. Never secure restraint straps to side rails. Always secure tie straps and buckles out of reach of the patient. For more information, consult your Posey District Sales Manager or Customer Service at 1.800.44.POSEY.

To request a Posey Catalog or speak with an associate, call 1.800.44.POSEY (76739) or visit us online at Posey.com

Posey Company • 5635 Peck Road, Arcadia, CA 91006-0020 USA • Phone: 1.800.447.6739 or 1.626.443.3143
Fax: 1.800.767.3933 or 1.626.443.5014 • www.posey.com ©2010 Posey Company. All rights reserved. MK1520 REV A 021811

